



ZOMET  
Alon Shevut, Gush Etzion  
Israel 90433  
Tel: (02) 9931442  
Fax: (02) 9931889  
Email: mail@zomet.org

**THE SHABBAT AMIGO ELECTRIC SCOOTER**

**HALACHIC APPROVAL**

This is to certify that Zomet's "Shabbat Control Unit" has been installed in this "AMIGO" Electric scooter, Serial#:

~~This special arrangement permits certain disabled persons (depending upon their specific circumstances) to use this scooter on Shabbat and Holidays in accordance with Halacha.~~

*Yisroel Rozen*  
Rabbi Yisroel Rozen, eng. Director of the Zomet Institute

Inspected, certified and signed by:

*Yonatan Lakfer*  
Yonatan Lakfer Eng.

Date: 2017 JAN 22

צוותי מדע ותורה  
אלון שבות גוש עציון 90433  
טל: 029931442

**INSTRUCTIONS FOR USE**

**Before Shabbat:**

Set the NORMAL/SHABBAT switch to the SHABBAT position. The Shabbat light will now flicker periodically. Note: This switch has a locking lever to prevent accidental switching to NORMAL mode and must be pulled out when changing position.

Use of lights, indicators and horn is not permitted on Shabbat and they are therefore disconnected.

**On Shabbat:**

In order to move the scooter, turn the key and wait. Within twenty seconds (after two flickers of the SHABBAT light) the scooter will turn on and begin to move very slowly (provided there is no heavy load or obstruction blocking its path). The scooter can now be driven using the accelerating lever.

To change direction (forward/backward) move the Shabbat directional switch and wait. The change will occur within twenty seconds (after two flickers of the SHABBAT light)

Upon reaching your destination, turn the key and wait. The scooter will shut off within twenty seconds (after two flickers of the SHABBAT light).

**Use of the Emergency Shabbat Stop Switch should be limited to Emergencies Only.**

The **brake release lever** located at the rear of the scooter may be used on Shabbat in order to push the scooter, however its **position may not be changed while the scooter is turned on.**


**Use of this scooter on Shabbat is only permitted in an area that has an ERUV. Please consult with local rabbinical authority in non ERUV areas.**

**After Shabbat:**

Set the NORMAL/SHABBAT switch to the NORMAL mode and if necessary recharge the battery.

s:\technology\tecdocs\wheelchair\amigo\amigo john approval+inst-english.doc

צוותי מדע ותורה

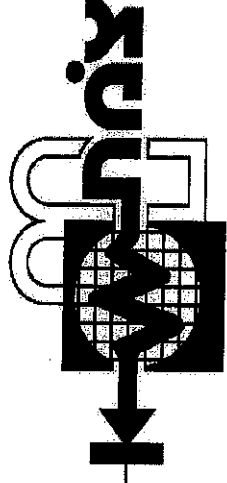

This is a specially modified

**SHABBOS AMIGO SCOOTER**

Designed and Authorized by

*The Zomet Institute - Israel*

 972-2-993-1442 • Fax: 972-2-993-1889

If the instructions supplied by The Zomet Institute are not strictly adhered to, Shabbos may be violated.

s:\technology\tecdocs\wheelchair'amigo\shitut-amigo-scooter-eng.doc